

Instructions for use

QuikRead[®] CRP






 Orion Diagnostics

Orion Corporation Orion Diagnostics
P.O.Box 83, 02101 Espoo, Finland
Tel. +358 10 42 995
Fax +358 10 429 2794
www.oriondiagnostics.fi

 ISO9001
CERTIFIED



1 Dispense 1 ml of buffer into the cuvette.

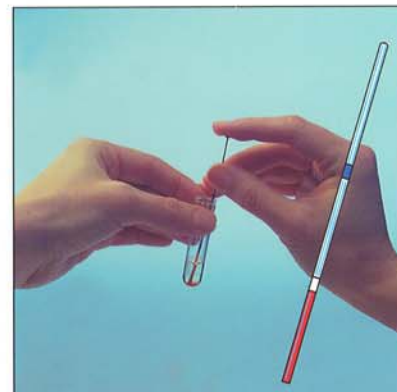

2 Add 20 µl of whole blood.




3 Put the cap on and mix gently (not upside-down). Put the cuvette in the measurement well.



4 Blank measurement max. 40 seconds. Add the CRP-reagent by pressing down the inner part of the cap.



5 Take out the cuvette and mix vigorously (back and forth) to dissolve the reagent.

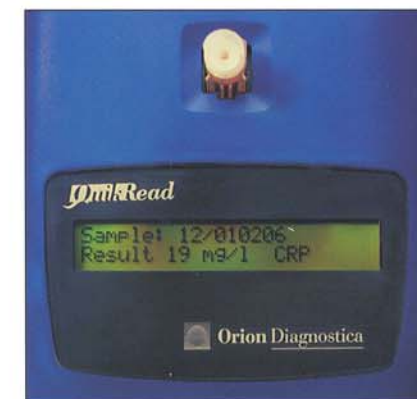

6 Put the cuvette back in the measurement well. Read the CRP result within 2 minutes.